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Multifuncional
Multifuncional Trocar Balloon

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The balloons of the Trocar device are supplied in packaging with 5 units

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Research, Development and Innovation

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The Trocar Balloon Dissector is a reusable instrumental device (permanent use), except for the balloon that is for single use.

The device allows the monitoring of the dissection of the anatomical space through the video-endoscopic optics inserted into the cannula inside the balloon, since not only is it transparent, but it also expands, allowing the presence of optics.

The Trocar Balloon is multifunctional, as it can be used in five different surgical procedures: such as common access sheath/trocarter for video-laparoscopic surgeries, trocar balloon dissector to create preperitoneal space in video-endoscopic surgery, in surgical procedures in the retroperitoneum, or as a trocar balloon dilator for laparoscopic nephrectomy and laparoscopic adrenalectomy using extraperitoneal techniques.

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Characteristics of the Multifunctional Trocar Balloon Device

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Trocar

Made from stainless steel, with a blunt tip, with 19 mm diameter and 15 cm length.

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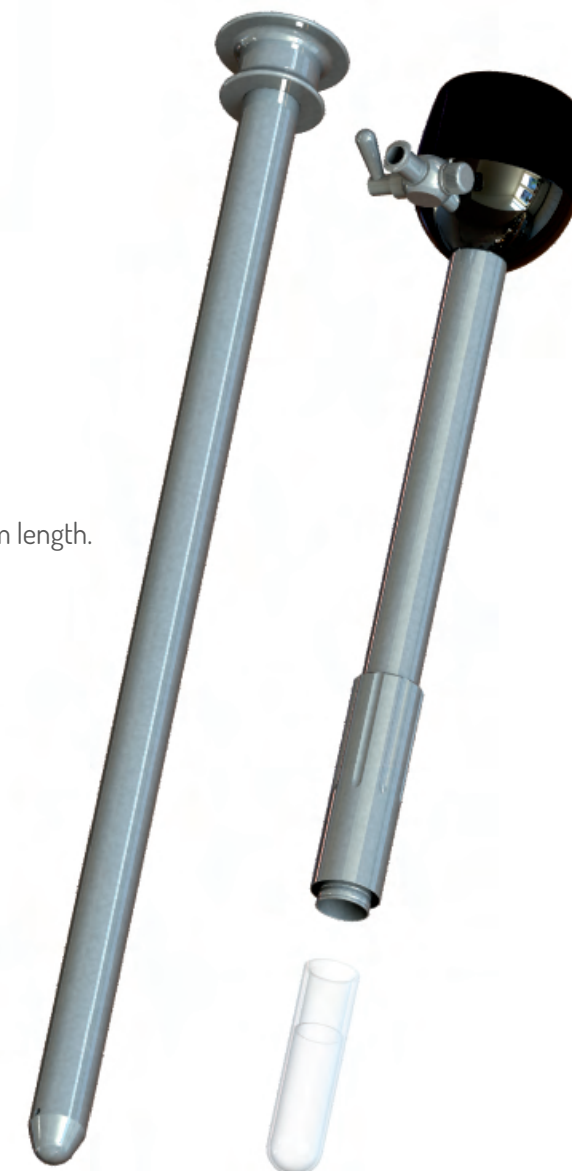
Sheath

Designed to work with a Ø 10mm endoscope, the sheath has a useful length of 15cm, and a bi-directional valve to facilitate the filling and emptying of the balloon. It also has a diaphragm-type silicone seal to protect

instrument access, which guarantees the seal during endoscope access (external seal), and without the instrument (internal seal).

It is manufactured out of Aluminum and Stainless steel, ensuring that the instrument is light and durable.

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Dissector Balloon

Single use balloons, made from medical grade high transparency silicone. Available in cylindrical and pear type models (tapered at the ends), and can reach up to 114mm in length by 66mm in diameter. The balloons are supplied non-sterile (sterilizable by autoclave process) purchased in kits with 5 units or as singles. Instrument is light and durable.